

Prism Cressy Sling



User Manual

PRISM MEDICAL UK

Prism Cressy Sling

The Prism Cressy sling has been designed in conjunction with the M&H team at Derbyshire CC. The continuous shoulder to hip strap allows flexibility in movement should a spasm occur and the person goes into abduction.

Application

The Cressy Sling provides maximum support to the sacral area. It can be used to transfer from the seated to recumbent position or from recumbent to seated. The person must be assessed by a competent person who must be fully trained in the suitability, application and fitting of the sling. Carers should always be trained in the use of hoists and slings.

Sling Inspection

All Prism Slings are manufactured to meet and exceed the requirements of BS EN 10535:2006 'Hoists for the transfer of disabled persons – Requirements and test methods'. Slings should be inspected prior to use and after washing. Damaged or badly worn slings should be discarded. The label on the sling contains vital information to identify the sling - if any part of the label becomes illegible, then it must be removed from service and replaced.

The sling should be visually inspected with regard to the following items:-

- Any signs of damage (i.e. Cuts, frays, tears, burns) to the sling straps
- Any signs of stitching becoming undone on any part of the sling
- Any signs of the fabric being stretched, tearing or undue wear
- Any signs of discolouring or bleaching on the material
- Is the sling complete? i.e. all straps present, stiffeners in place where applicable, etc
- Label clearly legible

Compatibility

The issue of sling compatibility with various manufacturers' hoists has been a concern for specifiers for a number of years, an issue that is compounded by the increased number of companies supplying hoists and slings.

Some hoist/sling manufacturers will state that only slings manufactured by them can be used on their hoists and vice versa. This is their policy only and is **NOT** a legal requirement. A risk assessment must still be completed even if the same ufacturer of hoist and sling are to be used, and the specifier simply has to state that the carry bar is of a similar design and that the sling manufacturer has its use on it.


A comprehensive list is available separately and your local agent will be able to provide assistance with compatibility issues.

Maximum Load (Safe Working Load - SWL)

The maximum load for the sling is clearly marked on the label of the sling. The maximum load will also be displayed on the hoist and any detachable elements of the hoisting system.

DO NOT exceed the maximum load stated on any of the equipment.

Sling Dimensions

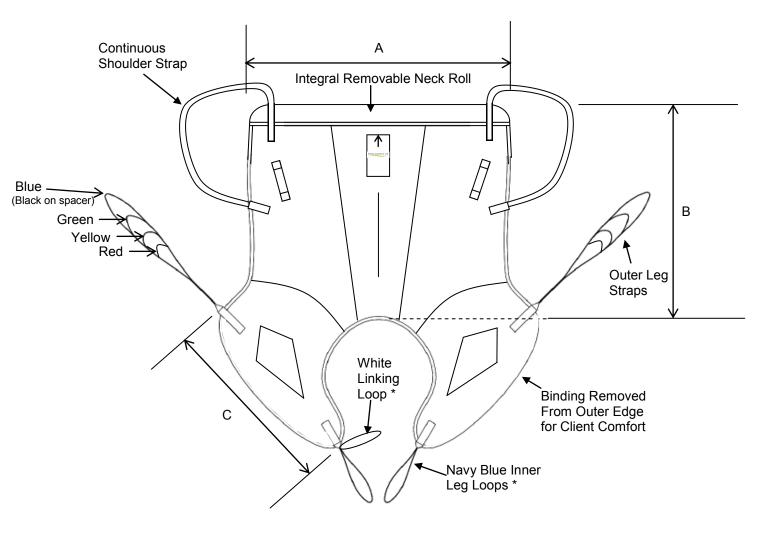

* These loops must NOT be attached directly to the carry bar on the hoist

	Α	В	С
Small	680	660	600
Medium	750	680	620
Large	810	730	650

All dimensions have a tolerance of +/- 20mm

Hoisting from a chair

Note: The person to be transferred shall be referred to as "the person".



Introduce the sling by sliding it down between the chair backrest and the the spine. Leave the sling half way down the persons back (don't drape over the persons shoulders)

Bring the leg sections along-side the person's thighs and pull lightly to ensure person's back until the top of the that they are of equal length. Note the commode aperture reaches the base of person has been asked to reach across to raise hip.



Using good manual handling techniques, raise the persons leg and feed the leg sections under the person's thighs. Ensure that the leg material is flat without any creases.



Both leg sections should be the same length and come through in the same position on the clients thighs.



Pass the Navy Blue Inner Leg Loop Pass the Outer Leg Straps through the through the White Linking Loop. Pass the Outer Leg Straps through the Inner Leg Loops.



Attach the Outer Leg Straps and the	Raise the carry bar just enough to				
Shoulder Straps to the Carry Bar.	tension the straps and ensure that they				
NOTE: The Inner leg loops and white	are still securely attached.				
linking loop must NOT be attached to					
the carry bar	The person can now be raised and				
	transferred.				

To remove the sling, carry out the fitting procedure in reverse. Important - always peel the sling away from the person to avoid skin friction and movement.

<u>Notes</u>

- The above techniques are for guidance only and you must comply with your local policies
- If in doubt, consult your moving and handling advisor

Hoisting from a lying position



Using good manual handling techniques roll the person towards you so that they are positioned on their side. introduce the sling, roll half of the sling without any creases. Repeat to both up underneath itself so that when the person is rolled onto their opposite side. the sling can be unrolled until it is flat. Use the centre rear panel on the back Pass the Navy Blue Inner Leg Loop section to ensure that the sling is in line with the person's spine. Ensure that handle loops are to the outside.




Raise the persons leg (if possible) and feed the leg sections under the person's To thighs. Ensure that the leg material is flat sides. Both leg sections should be the same length and come through in the same position on the clients thighs. through the White Linking Loop.





Pass the Outer Leg Straps through the If using a profile bed, profile the bed to a Inner Leg Loops. sitting position. Ease the back section upwards to ensure that there are no creases in the back section.

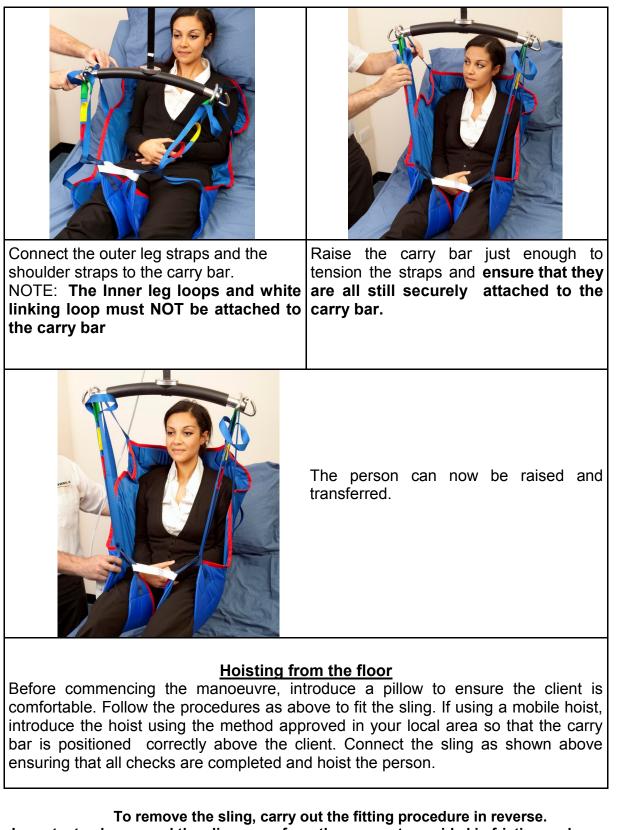


Important - always peel the sling away from the person to avoid skin friction and movement.

<u>Notes</u>

- The above techniques are for guidance only and you must comply with your local policies
- If in doubt, consult your moving and handling advisor

Sling position

The persons position can be adjusted by simply using a different coloured loop on the outer leg straps. The continuous shoulder strap will self adjust.

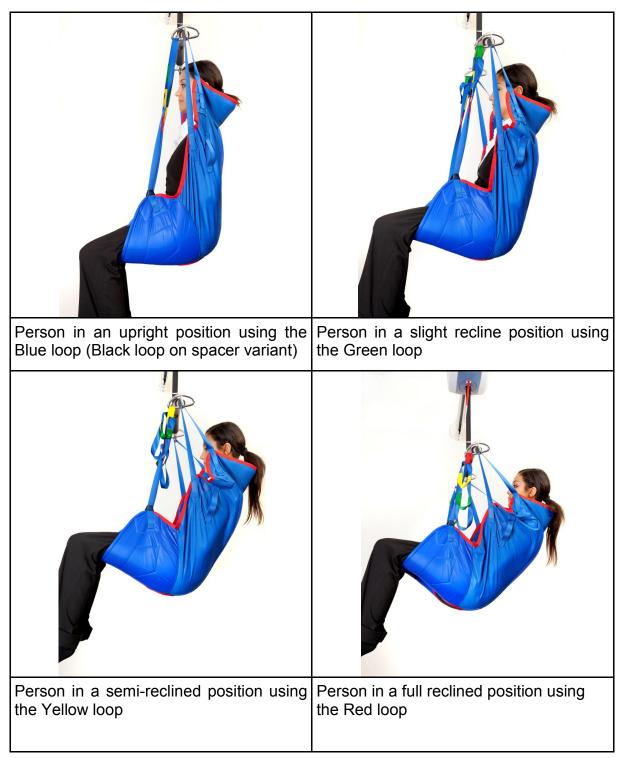

Prism Cressy Sling - Material Variants / Uses

Standard Slings

- **Poly:** The sling is made from a close knit polyester material and has nylon 'rip stop' material to the outer side of the leg sections to aid the fitting of the sling. This is a general purpose sling and can be used for sitting and Recumbent transfers between chairs, beds, baths, changing beds, WC's etc.
- **Spacer:** The sling is made from knitted polyester "spacer" fabric that is manufactured with an 'air gap' internally. The material stretches slightly in all directions and moulds perfectly to the shape of the person being hoisted. This is a general purpose sling that can be used as the Poly but has the added benefit of being suitable to be left in position if required (subject to risk assessment and monitoring of pressure risks).

SLING CARE: LAUNDERING

GENERAL.

Sensitive laundering of lifting slings will extend the service life of the product. Conversely, improper laundering will reduce the service life and possibly affect the strength and integrity of the sling.

To be avoided:

- Use of bleach at washing, particularly hypochlorite [may cause structural damage]
- Overheating the sling [temperatures in excess of 100° may cause extreme shrinkage and structural damage]

Use of non-biological detergents is not recommended (especially Fairy Non - Bio): as these types of detergent contain bleaching agents that can degrade the strength of the sling.

The use of fabric conditioners is not recommended.

Any hook & loop fastenings should be closed prior to laundering to avoid the 'hook' side damaging the sling or other items laundered with it.

GENERAL SOILING.

The sling should be laundered with soap or detergent at 50°C on a standard domestic cycle.

DISINFECTION.

The sling should be laundered to ensure a 'period at temperature' of:

- 10 minutes at 65°C
- 3 minutes at 72°C

Following NHS guidelines, extra time should be added to ensure thorough mixing of hot water through the wash load.

It may be that available equipment does not allow the required level of temperature control. The sling may be washed on a 90°C wash cycle, though it is likely that some moderate shrinkage and/or colour bleed will occur.

DRYING.

Slings will air dry quite quickly in a warm room.

Slings should not be placed over radiators or convectors.

Slings may be **CAREFULLY** tumble dried on a cool [60°C] setting and short cycle. The sling should be checked at intervals not exceeding 5 minutes.

Do not allow the sling to overheat.

The sling should last in excess of 5 years if the above guidelines are adhered to. All Prism slings have a manufacturers guarantee of 12 months.

DECLARATION OF CONFORMITY				
Manufacturers Name:	Prism Medical UK Ltd			
MHRA Registration No:	CA 013248			
Manufacturers Address:	Unit 4 Jubilee Business Park, Jubilee Way,			
	Grange Moor, Wakefield,			
	WF4 4TD, UK			
	Tel +44 (0) 844 980 2260			
Declares that the manufactur	red product:-			
Product Name:	Prism Cressy Sling.			
Product Options :	Small, Medium, Large,			
Conforms to the following E	European Union Council Directives:-			
Directive 93/42/EEC	and all amendments up to Directive 2007/47/EC			
Classification	on Class 1, Low Risk Medical Device			
Tested in ac	ecordance with BS EN ISO 10535:2006 & IEC 60601 3rd Edition			

Supplementary Information: The undersigned declares the product herewith complies with the requirements set out above and carries the CE mark accordingly.

The Technical Construction File required by this directive is maintained by the manufacturer as detailed above.

Signed:

×

Craig Wright, Manufacturing Director

Date: 14

14th November 2014

© Prism Medical UK 2015

Care Plan Information

All Prism Cressy Slings have space on the labels to record the clients name. Each sling has unique individually colour coded lifting loops on the outer leg straps which helps maintain good lifting practice. The loop options facilitate variable positions from sitting to recline.

Date of Assessment :			FIM Score:					
Clients Name:								
Approx Weight:			D.O.B:					
Type of Sling:			Size:					
	Sling Straps Coloured Loop Options							
Outer Leg Strap	Blue/ Black 🛛	Green 🗆	Yellov	w 🗆 🛛 Red				
Inner Leg Loop	Loop Position 1		Lo	Loop Position 2				
Linking Loop	Loop Position 1		Loop Position 2 \Box					
	Comments							

If you have any questions regarding the use of this equipment, please contact Prism Medical UK, or your local authorised dealer.



Prism Medical UK, Unit 4 Jubilee Business Park, Jubilee Way, Grange Moor, Wakefield, WF4 4TD

Tel +44 (0) 844 980 2260

www.prismmedical.co.uk

info@prismmedical.co.uk

This document conforms to BS EN ISO 10535:2006 Requirements